
FocusTB

Towards a TB free world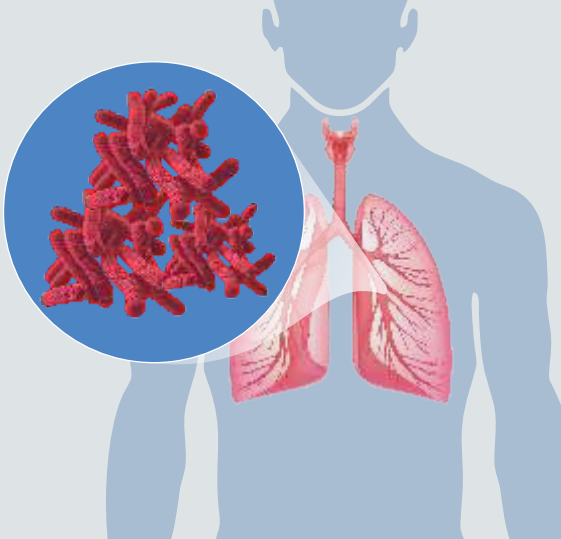

TGS

TB - Whole Genome Sequencing

18 drug susceptibility profile | 12 Days TAT

@ ₹ 15,000
6000* /-

**Offer valid for limited period only.*


TO AVAIL SERVICE

An initiative by

Thyrcare[®]

The Trust. The Truth.

Powered by

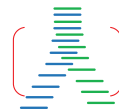HaystackAnalytics

How to collect the sputum sample?

Take a clean Sterile Specimen Collection Cup with a Lid - easily available in any nearby medical store

As soon as you wake up in the morning (before you eat or drink anything), rinse your mouth with water Do not use mouthwash.

Take a very deep breath and hold the air for 5 seconds. Slowly breathe out. Take another deep breath and cough hard until some sputum comes up into your mouth.

Spit the sputum into the plastic cup.

Keep doing this until the sputum reaches the 5 ml line (or more) on the plastic cup.

Screw the cap on the cup tightly so it doesn't leak

Give your sample to your caregiver immediately

If you are at home, put your sample in a plastic bag. Close the bag and put in the fridge right away. Handover the sample to your caregiver within 24 hours

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2. Open the sample container.


3. Hold the container to your hand.

4. Take as deep a breath as you can and cough then spit into the container (do NOT just spit saliva).

5. The sample you cough should look thick and yellow or green. More than a tablespoon of sample is needed.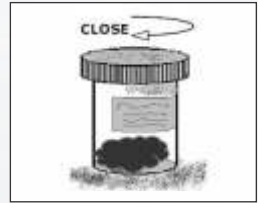

6. Close the container lid tightly and seal with parafilm.

7. Give the sample to your caregiver right away

8. If you are at home:

- Put your sample in the plastic bag you were given.
- Close the bag and put it in the fridge right away.
- Return your sample to your caregiver within 24 hours.